RadiForce® MS230W

Color LCD Monitor

English

Deutsch

Français

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Setup Manual

Important: Please read PRECAUTIONS, this Setup Manual and the User's Manual stored on the CD-ROM carefully to familiarize yourself with safe and effective usage. Please retain this manual for future reference.

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Color LCD Monitor



Package Contents

Check that all the following items are included in the packaging box. If any items are missing or damaged, contact your local dealer.


NOTE

Please keep the packaging box and materials for future movement or transport of the monitor.

- Monitor
- Power cord
- Digital signal cable (DVI) : FD-C39

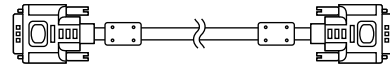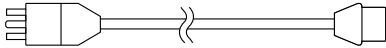
- Digital signal cable (DisplayPort) : PP200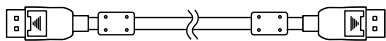

- EIZO LCD Utility Disk (CD-ROM)
- User's Manual
- Setup Manual (this manual)
- Mounting Screws (M4 x 12mm, 4 pcs)
- PRECAUTIONS
- Recycling Information

Controls and Functions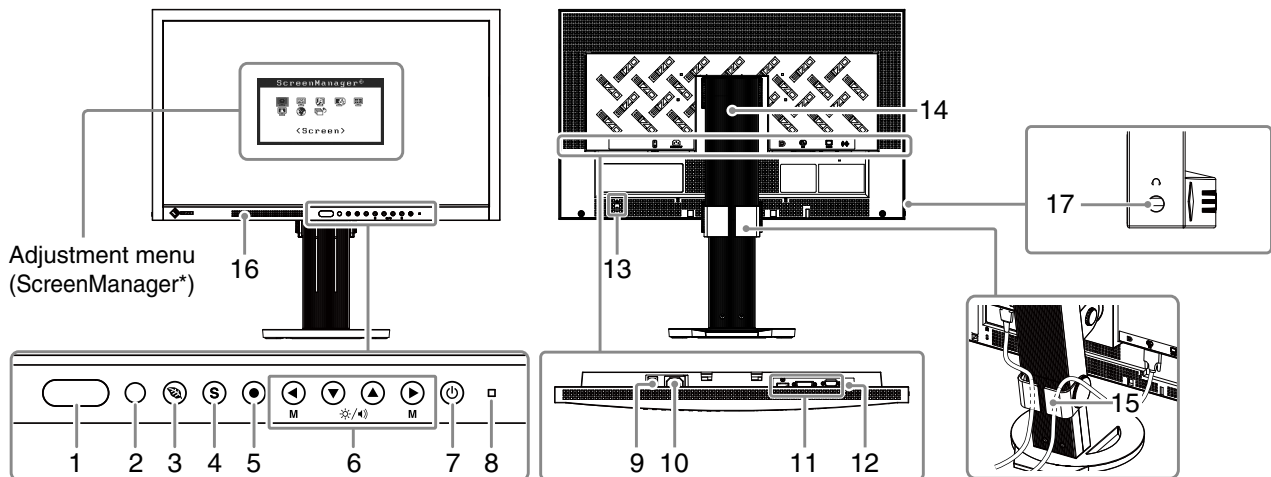

1	EcoView Sense sensor	Detects the movement of a person in front of the monitor. EcoView Sense function.
2	Auto EcoView sensor	Detects ambient brightness. Auto EcoView function.
3	Ⓢ button	Displays the setting menu of the power saving function (Auto EcoView, EcoView Sense and EcoView Index) .
4	Ⓢ button	Switches input signals for display.
5	Ⓢ button	Displays the Adjustment menu, determines an item on the menu screen, and saves values adjusted.
6	⏪ ⏩ ⏴ ⏵ button	<ul style="list-style-type: none"> • Chooses an adjustment item or increases/decreases adjusted values for advanced adjustments using the Adjustment menu. • ⏴ or ⏵ button: Allows you to switch the CAL Switch mode. • ⏪ or ⏩ button: Displays the adjustment menu of the brightness and volume.
7	Ⓢ button	Turns the power on or off.
8	Power indicator	Indicates monitor's operation status. Blue: Operating Orange: Power saving Off: Main Power / Power off
9	Main Power Switch	Turns the main power on or off.
10	Power connector	Connects the power cord.
11	Input signal connectors	Left : DisplayPort Connector / Center : DVI-D Connector / Right : D-Sub mini 15-pin connector
12	Stereo mini jack	Connects the stereo mini jack cable.
13	Security lock slot	Complies with Kensington's MicroSaver security system.
14	Stand	Used to adjust the height and angle of the monitor screen.
15	Cable holder	Covers the monitor cables.
16	Speaker	Outputs audio source.
17	Headphone jack	Connects the headphones.

* ScreenManager is an EIZO's nickname of the Adjustment menu. For how to use ScreenManager, refer to the User's Manual on the CD-ROM.

Connecting Cables

Note


- Check that the monitor and the PC are powered off.
- When replacing the current monitor with a MS230W monitor, be sure to change the PC settings for resolution and vertical frequency to those which are available for the MS230W monitor referring to the resolution table (back of cover page) before connecting the PC.

Tips

- When connecting multiple PCs to the monitor, refer to the User's Manual on the CD-ROM.

1 Connect the monitor to the PC with a signal cable that matches the connectors.

After connecting the cable connectors, tighten the screws of the connectors to secure the coupling.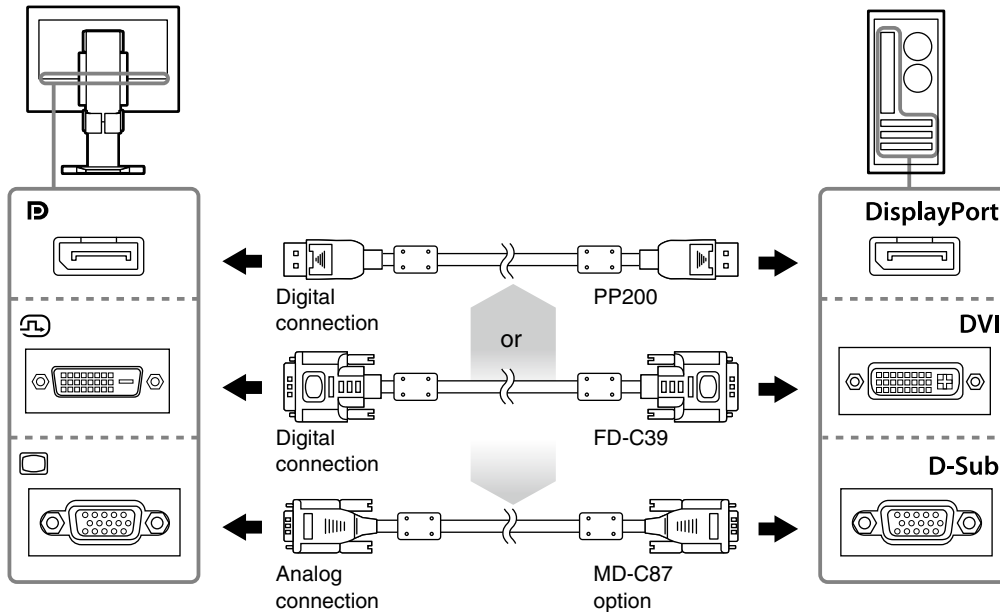

2 Plug the power cord into a power outlet and the power connector on the monitor.

3 Press to turn on the monitor.

The monitor's power indicator lights up blue.

4 Turn on the PC.

The screen image appears.


If an image does not appear, refer to "No-Picture Problem" (page 4) for additional advice.

Note

- Turn off the monitor and PC after using them.
- For the maximum power saving, it is recommended that the Power button be turned off. Turning off the main power switch or unplugging the power cord completely shuts off power supply to the monitor.

Adjusting the Screen Height and Angle

Hold left and right edge of the monitor with both hands, and adjust the screen height, tilt and swivel of the screen to the best condition for working.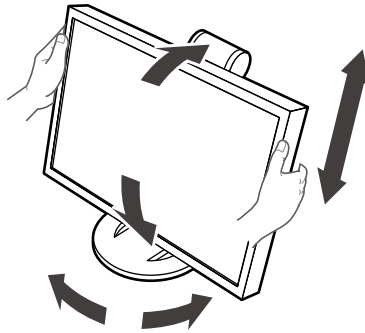

Note


- Be sure that the cables are correctly connected.

Selecting Display Mode

This function allows easy selection of a desired mode according to monitor application.

Display Modes

Mode	Purpose
Custom	Available for making desired setting.
sRGB	Suitable for color matching with sRGB compatible peripherals.
Text	Suitable for displaying texts for word processing or spreadsheets.
DICOM	Available to display in DICOM mode.
Movie	Suitable for playing back animated images.

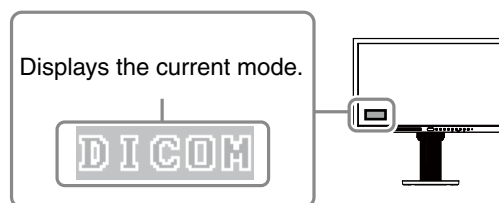- 1 Press ◀ or ▶.
A mode name appears at the lower left of the screen.
- 2 Change the mode by pressing ◀ or ▶ while the mode name is displayed.

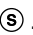
Tips

- The Adjustment menu and the Mode menu cannot be displayed at the same time.

No-Picture Problem


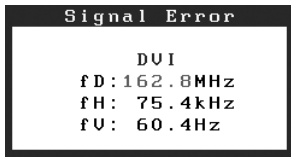
If no picture is displayed on the monitor even after the following remedial action is taken, contact your local dealer.

1. Check the Power indicator.

Symptom	Status	Possible cause and remedy
No picture	Power indicator does not light.	<ul style="list-style-type: none"> • Check whether the power cord is connected correctly. • Turn off the main power, and then turn it on again a few minutes later. • Turn the main power switch on. • Press .
	Power indicator lights blue.	<ul style="list-style-type: none"> • Set each adjusting value in [Brightness], [Contrast] and [Gain] to higher level.
	Power indicator lights orange.	<ul style="list-style-type: none"> • Switch the input signal with . • Operate the mouse or keyboard. • Check whether the PC is turned on. • When the EcoView Sense is on, the monitor may have shifted to the power saving mode. Come near to the monitor.
	Power indicator flashes orange.	<ul style="list-style-type: none"> • There is a problem in the device that uses DisplayPort connection. Solve the problem, turn off the monitor, and then turn it on again. Refer to the manual of the device connected to the DisplayPort for details.

2. Check the error message that remains on the screen.

These messages appear when the input signal is incorrect even if the monitor is functioning.

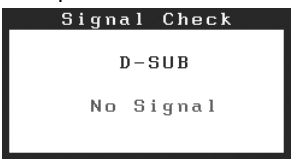Symptom	Status	Possible cause and remedy
Example: 	This message appears when no signal is input.	<ul style="list-style-type: none"> • Check whether the PC is turned on. • Check whether the signal cable is properly connected to the PC. • Switch the input signal with .
Example: 	The message shows that the input signal is out of the specified frequency range. (Such signal frequency is displayed in red.)	<ul style="list-style-type: none"> • Check whether the signal setting of your PC matches the resolution and the vertical frequency settings for the monitor. • Reboot the PC. • Select an appropriate display mode using the graphics board's utility. Refer to the manual of the graphics board for details. fD: Dot Clock (Displayed only during digital signal input) fH: Horizontal Frequency fV: Vertical Frequency

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
[Location of the Caution Statements]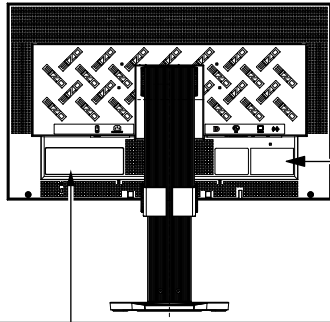

The equipment must be connected to a grounded main outlet.
Jordet stikkontakt skal benyttes når apparatet tilkobles datanett.
Apparaten skall anslutas till jordat nätuttag.
这设备必须连接至接地主插座。
電源コードのアースは必ず接地してください。

CAUTION
Risk of electric shock. Do not open.

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No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation.
EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.