# Installation Manual

## RadiForce® GX540-CL

Monochrome LCD monitor

### Important

Please read this Installation Manual, and “Instructions for Use” (separate volume) carefully to familiarize yourself with safe and effective usage.

- 
- Please refer to the “Instructions for Use” for basic information ranging from connection of the monitor to a PC to using the monitor.
  - The latest Installaion Manual is available for download from our web site:  
<http://www.eizoglobal.com>
- 

This product has been adjusted specifically for use in the region to which it was originally shipped. If operated outside this region, the product may not perform as stated in the specifications.

---

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation.

EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

---

# CONTENTS


<b>CONTENTS</b> .....	<b>3</b>	<b>Chapter 4 Controlling Monitor Quality</b> .....	<b>18</b>
<b>Chapter 1 Basic Operations and Functions</b> ....	<b>4</b>	4-1. Executing Calibration.....	18
1-1. Basic Adjustment menu operations .....	4	● Executing SelfCalibration.....	18
1-2. Functions .....	5	● Viewing SelfCalibration result.....	19
<b>Chapter 2 Adjusting Screens</b> .....	<b>7</b>	4-2. Checking Grayscale.....	19
2-1. Compatible Resolutions .....	7	● Executing Grayscale Check.....	19
2-2. Setting the Resolution .....	7	● Viewing grayscale check results.....	19
● Windows 8 / Windows 7 .....	7	4-3. Acquiring Life Prediction Data .....	20
● Windows Vista.....	7	4-4. Setting On/Off for Warning and QC	
● Windows XP .....	7	History.....	20
● Mac OS X.....	7	4-5. Using Quick Image Checker function.....	20
2-3. Adjusting Mode .....	8	<b>Chapter 5 Power Saving Functions</b> .....	<b>21</b>
● Selecting the display mode		5-1. Setting Power Save.....	21
(CAL Switch mode) .....	8	5-2. Adjusting the Brightness of Indicator .....	21
● Adjustable settings in each mode .....	9	5-3. Setting DisplayPort Power Save .....	22
● Adjusting Brightness .....	9	5-4. Setting the Power Saving Mode when a	
● Setting the mode in the monochrome		Person Leaves the Monitor.....	23
image display area .....	10	<b>Chapter 6 Troubleshooting</b> .....	<b>24</b>
● Setting the mode in the base display area.....	10	6-1. No Picture .....	24
● Setting the detection type in the image		6-2. Imaging Problems.....	25
display area .....	11	6-3. Other Problems .....	26
● Setting the highlight area in the image		<b>Chapter 7 Reference</b> .....	<b>27</b>
display area .....	11	7-1. Attaching Arm .....	27
● Setting the sensitivity level for detecting		7-2. Connecting Multiple PCs .....	28
the ambient illuminance .....	11	● Switching among input signals.....	28
● Setting ambient illuminance and related		● Setting input signal switching procedure .....	29
settings for Lamb.....	12	7-3. Making Use of USB	
● Displaying the current value of the		(Universal Serial Bus).....	30
ambient luminance .....	12	● Required system environment .....	30
<b>Chapter 3 Setting Monitor</b> .....	<b>13</b>	● Connection procedure	
3-1. Setting Orientation .....	13	(Setup of USB function).....	30
3-2. Showing and hiding the Grayscale		7-4. Showing Monitor Information.....	31
Display Error .....	13	● Displaying signal information .....	31
3-3. Setting Input Signal .....	14	● Showing monitor information .....	31
● To connect DVI.....	14	7-5. Main Default Settings .....	32
● To connect DisplayPort.....	14	<b>Appendix</b> .....	<b>33</b>
3-4. Enabling/Disabling Mode Selection .....	15	LIMITED WARRANTY.....	33
3-5. Setting Language.....	15	Recycling Information .....	40
3-6. Setting Adjustment Menu Display .....	15		
3-7. Showing and hiding the EIZO logo .....	16		
3-8. Locking Operation Buttons .....	16		
3-9. Enabling/Disabling DDC Communication...16			
3-10. Enabling/Disabling DC5V Output.....	17		
3-11. Resetting to Default.....	17		
● Resetting the mode adjustments .....	17		
● Resetting all adjustments to the default			
settings.....	17		

# Chapter 1 Basic Operations and Functions

## 1-1. Basic Adjustment menu operations

### 1. Displaying Adjustment menu

1. Press ENTER. The Adjustment menu appears.

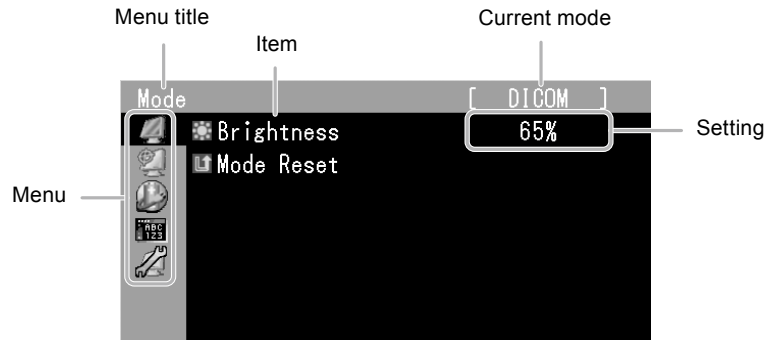### 2. Adjusting/Setting

1. Select a menu to adjust/set with ▲ or ▼, and then press ENTER.

2. Select an item to adjust/set with ▲ or ▼, and then press ENTER.

3. Adjust/set the selected item with ▲ or ▼, and then press ENTER.

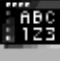
### 3. Exiting the software

1. Press RETURN a few times. The Adjustment menu finishes.

## 1-2. Functions

The following table shows the Adjustment menu and setting items of each menu.

Main Menu	Item	Reference
Mode 	 Brightness	"2-3. Adjusting Mode" (page 8)
	 Gray Area	
	 Base Area	
	 Hybrid Assignment	
	 Highlight Area	
	 Sensitivity	
	 Lamb Target (Low)	
	 Lamb Target (Std.)	
	 Lamb Target (High)	
	 Current Status	"3-11. Resetting to Default" (page 17)
 Mode Reset		
RadiCS SelfQC 	 SelfCalibration	"4-1. Executing Calibration" (page 18)
	 Grayscale Check	"4-2. Checking Grayscale" (page 19)
	 LEA	"4-3. Acquiring Life Prediction Data" (page 20)
	 Settings	"4-4. Setting On/Off for Warning and QC History" (page 20)
Power Manager 	 Power Save	"5-1. Setting Power Save" (page 21)
	 Indicator	"5-2. Adjusting the Brightness of Indicator" (page 21)
	 Presence Sensor	"5-4. Setting the Power Saving Mode when a Person Leaves the Monitor" (page 23)
Menu Settings 	 Language	"3-5. Setting Language" (page 15)
	 Menu Position	"3-6. Setting Adjustment Menu Display" (page 15)
Tools 	 Input	"Switching among input signals" (page 28)
	 Input Selection	"Setting input signal switching procedure" (page 29)
	 Mode Preset	"3-4. Enabling/Disabling Mode Selection" (page 15)
	 Signal Info	"7-4. Showing Monitor Information" (page 31)
	 Monitor Info	
	 All Reset	"3-11. Resetting to Default" (page 17)

Main Menu	Item		Reference
Optional Settings	Logo		<a href="#">“3-7. Showing and hiding the EIZO logo” (page 16)</a>
	Key Lock		<a href="#">“3-8. Locking Operation Buttons” (page 16)</a>
	Orientation		<a href="#">“3-1. Setting Orientation” (page 13)</a>
	Grayscale Warning		<a href="#">“3-2. Showing and hiding the Grayscale Display Error” (page 13)</a>
	Input Signal		<a href="#">“3-3. Setting Input Signal” (page 14)</a>
	Connection Config.	DDC	<a href="#">“3-9. Enabling/Disabling DDC Communication” (page 16)</a>
		DC5V Output	<a href="#">“3-10. Enabling/Disabling DC5V Output” (page 17)</a>
DP Power Save		<a href="#">“5-3. Setting DisplayPort Power Save” (page 22)</a>	

# Chapter 2 Adjusting Screens

## 2-1. Compatible Resolutions

For details on compatible resolutions, refer to “Compatible Resolutions” in the “Instructions for Use”.

## 2-2. Setting the Resolution

### ● Windows 8 / Windows 7

1. For Windows 8, click the “Desktop” tile on the Start Screen to display the desktop.
2. Right-click the mouse anywhere on the desktop except for icons.
3. From the displayed menu, click “Screen resolution”.
4. On the “Screen Resolution” dialog box, select the monitor.
5. Click “Resolution” to select the desired resolution.
6. Click the “OK” button.
7. When a confirmation dialog box is displayed, click “Keep changes”.

### ● Windows Vista

1. Right-click the mouse anywhere on the desktop except for icons.
2. From the displayed menu, click “Personalize”.
3. On the “Personalization” window, click “Display Settings”.
4. On the “Display Settings” dialog box, select the “Monitor” tab and select desired resolution in the “Resolution” field.
5. Click the “OK” button.
6. When a confirmation dialog box is displayed, click “Yes”.

### ● Windows XP

1. Right-click the mouse anywhere on the desktop except for icons.
2. From the displayed menu, click “Properties”.
3. When the “Display Properties” dialog box is displayed, click the “Settings” tab and select desired resolution for “Screen resolution” under “Display”.
4. Click the “OK” button to close the dialog box.

### ● Mac OS X

1. Select “System Preferences” from the Apple menu.
2. When the “System Preferences” dialog box is displayed, click “Displays” for “Hardware”.
3. On the displayed dialog box, select the “Display” tab and select desired resolution in the “Resolutions” field.
4. Your selection will be reflected immediately. When you are satisfied with the selected resolution, close the window.

## 2-3. Adjusting Mode

### ● Selecting the display mode (CAL Switch mode)


A desired mode can be selected easily according to the monitor application.

#### Display Modes

√: Calibration is available.

Mode	Purpose	
1-DICOM	Selected for the display in the DICOM mode.	√
2-CAL1 / 3-CAL2	Displays the screen adjusted by calibration software.	√
4-Hybrid-γ	Automatically identifies the display area for the medical and other images on the same screen, and displays each set status.	-
5-ALT <sup>*1</sup>	Suitable for displaying the images with the brightness and grayscale appropriate for the ambient illuminance.	-

\*1 The screen may not display properly depending on the usage environment. The quality of the monitor cannot be managed using this mode.

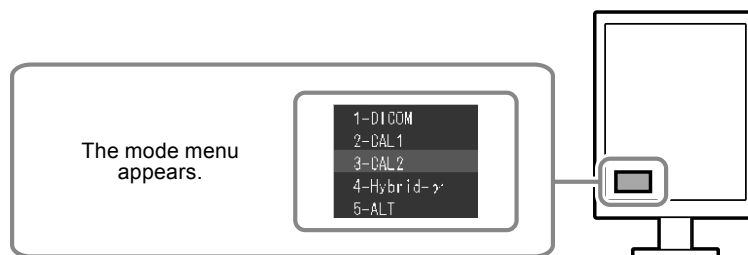#### Procedure

1. Press MODE.

The mode menu appears at the bottom left of the screen.

2. The mode among the list is highlighted in turn each time MODE is pressed.

You can switch the mode with ▲ or ▼ while the mode menu is displayed.

#### Note

- The Adjustment menu and the Mode menu cannot be displayed at the same time.

## ● Adjustable settings in each mode

The adjustable settings depend on the mode. (Non-adjustable or non-settable function cannot be selected.)

√: Adjustable -: Not adjustable

Icon	Function	CAL Switch mode			
		1-DICOM	2-CAL1 3-CAL2	4-Hybrid-y	5-ALT
	Brightness	√	√	-	-
	Gray Area	-	-	√	-
	Base Area	-	-	√	-
	Hybrid Assignment	-	-	√	-
	Highlight Area	-	-	√	-
	Sensitivity	-	-	-	√
	Lamb Target (Low)	-	-	-	√
	Lamb Target (Std.)	-	-	-	√
	Lamb Target (High)	-	-	-	√
	Mode Reset	√	√	√	√

## ● Adjusting Brightness

The screen brightness is adjusted by changing the brightness of the backlight (light source from the LCD back panel).

### **Setting Range**

0 to 100%

### **Procedure**

1. Select "Mode" from the Adjustment menu, and press ENTER.
2. Select "Brightness" from "Mode", and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

## ● Setting the mode in the monochrome image display area

The mode suitable for the monochrome image display area can be set (see “[Selecting the display mode \(CAL Switch mode\)](#)” (page 8)).

### **Setting Range**

DICOM, CAL1, CAL2

### **Procedure**

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Gray Area” from “Mode”, and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

---

#### **Attention**

- After the configuration in the display area, you cannot select any mode for which Mode Preset is “Off” (see “[3-4. Enabling/Disabling Mode Selection](#)” (page 15)).

---

#### **Note**

- The brightness is the same as the value for the mode selected. To change the brightness, use the mode selected for adjustment.
- 

## ● Setting the mode in the base display area

The mode suitable for the base display area can be set.

### **Setting Range**

Base-L, Base-H

### **Procedure**

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Base Area” from “Mode”, and press ENTER.
3. Select “Base-L” or “Base-H” with ▲ or ▼.

Mode	Purpose
Base-L	Specifies the target brightness to 100 cd/m <sup>2</sup> .
Base-H	Specifies the target brightness to 170 cd/m <sup>2</sup> .

4. Press ENTER to exit.

---

#### **Attention**

- After the configuration in the display area, you cannot select any mode for which Mode Preset is “Off” (see “[3-4. Enabling/Disabling Mode Selection](#)” (page 15)).

---

#### **Note**

- If the brightness value in the base display area is higher than the target brightness, the target brightness is applied for Base-L and Base-H.
-

## ● Setting the detection type in the image display area

The detection type in the display area can be set.

### **Procedure**

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Hybrid Assignment” from “Mode”, and press ENTER.
3. Select “Full Scan”, “Partial Scan”, or “Custom” with ▲ or ▼.

Detection Type	Purpose
Full Scan	Identifies the display area automatically.
Partial Scan	Identifies the selection area using software (Hybrid Gamma Manager) automatically.
Custom	Allows you to set the display area manually using software (Hybrid Gamma Manager).

4. Press ENTER to exit.

---

### **Note**

- For Hybrid Gamma Manager, contact your local EIZO representative.
- 

## ● Setting the highlight area in the image display area

On/Off for the highlight area in the image display area can be set.

### **Procedure**

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Highlight Area” from “Mode”, and press ENTER.
3. Select “On” or “Off” with ▲ or ▼.
4. Press ENTER to exit.

---

### **Note**

- You can set On/Off for the highlight area with ▼.
- 

## ● Setting the sensitivity level for detecting the ambient illuminance

The sensitivity level for detecting the ambient illuminance (running mean time) can be adjusted. The higher the sensitivity level is, the more frequently the brightness and LUT will be updated.

### **Setting Range**

Level 1 to level 5

### **Procedure**

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Sensitivity” from “Mode”, and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

## ● Setting ambient illuminance and related settings for Lamb

In the ALT mode, the brightness and LUT of the monitor display are changed according to the desired target value specified for the ambient luminance (Lamb). Three types of settings are available according to your environment.

Item	Purpose
Lamb Target (Std.)	The illuminance in the standard environment can be selected. Select the desired minimum luminance, and maximum luminance suitable for the illuminance.
Lamb Target (Low)	The illuminance in the darkest environment can be selected. Select the desired minimum luminance, and maximum luminance suitable for the illuminance.
Lamb Target (High)	The illuminance in the brightest environment can be selected. Select the desired minimum luminance, and maximum luminance suitable for the illuminance.

### Illuminance (lx)

#### **Procedure**

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Lamb Target (Low)”, “Lamb Target (Std.)”, or “Lamb Target (High)” from “Mode”, and press ENTER.
3. Select “Illuminance”, and press ENTER.
4. Use ▲ or ▼ for adjustment.
5. Press ENTER to exit.

### Minimum brightness value/maximum brightness value

#### **Procedure**

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Lamb Target (Low)”, “Lamb Target (Std.)”, or “Lamb Target (High)” from “Mode”, and press ENTER.
3. Select “Lmin” or “Lmax”, and press ENTER.
4. Use ▲ or ▼ for adjustment.
5. Press ENTER to exit.

## ● Displaying the current value of the ambient luminance

The current value of the ambient luminance can be displayed.

#### **Procedure**

1. Select “Mode” from the Adjustment menu, and press ENTER.
2. Select “Current Status” from “Mode”, and press ENTER.

The current values of the illuminance, minimum luminance, maximum luminance, and ambient luminance are displayed.

# Chapter 3 Setting Monitor

## 3-1. Setting Orientation

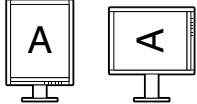
The display direction of the image can be changed according to your environment.

### Attention

- Refer to the User's Manual of the graphics board.
- The setting is available even when the PC is not started up.

### Procedure

1. Press  $\mathcal{O}$  to turn off the monitor.
2. Hold down MODE while pressing  $\mathcal{O}$  for at least two seconds to turn on the monitor.  
The "Optional Settings" menu appears.
3. Select "Orientation" from "Optional Settings", and press ENTER.
4. Select "Landscape", "Portrait(SW)", or "Portrait(HW)" with  $\blacktriangle$  or  $\blacktriangledown$ , and press ENTER.

Monitor orientation	Display example
Landscape Selected for horizontal installation.	
Portrait(SW) Selected for vertical installation. The display image is turned by 90° using the utility software of the graphics board.	
Portrait(HW) Selected for vertical installation. The display image is turned by 90° using the monitor function.	

5. Select "Finish" with  $\blacktriangle$  or  $\blacktriangledown$ .
6. Press ENTER.  
The "Optional Settings" menu is closed.

## 3-2. Showing and hiding the Grayscale Display Error

If a 6-bit (64 color) signal is detected when connecting to the DisplayPort, a grayscale display error message can be displayed. If the error message appears, please turn off the main power, and then turn it on again.

### Attention

- Always use with this set to "ON" (the default setting is "ON").
- Set this to "OFF" only when using 6-bit (64 grayscale) display.

### Procedure

1. Press  $\mathcal{O}$  to turn off the monitor.
2. Hold down MODE while pressing  $\mathcal{O}$  for at least two seconds to turn on the monitor.  
The "Optional Settings" menu appears.
3. Select "Grayscale Warning" from "Optional Settings", and press ENTER.
4. Select "On" or "Off" with  $\blacktriangle$  or  $\blacktriangledown$ , and press ENTER.
5. Select "Finish" with  $\blacktriangle$  or  $\blacktriangledown$ .
6. Press ENTER.  
The "Optional Settings" menu is closed.

## 3-3. Setting Input Signal

---

The settings for the input signal may be required depending on the usage environment. Signals can be selected based on your operating environment.

### **Attention**

- A supported viewer is required to display.
  - Setting is available even when the PC is not started up.
- 

### ● To connect DVI

#### **Procedure**

1. Press **⏻** to turn off the monitor.
2. Hold down **MODE** while pressing **⏻** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Signal Selection” from “Optional Settings”, and press **ENTER**.
4. Select “DVI” with **▲** or **▼**, and press **ENTER**.
5. Select “Signal Format” with **▲** or **▼**, and press **ENTER**.
6. Select “Standard” or “SDG 8” with **▲** or **▼**, and press **ENTER**.
7. Select “Sub Pixel Drive” with **▲** or **▼**, and press **ENTER**.
8. Select “On” or “Off” with **▲** or **▼**, and press **ENTER**.
9. Press **RETURN** twice.
10. Select “Finish” with **▲** or **▼**.
11. Press **ENTER**.  
The “Optional Settings” menu is closed.

### ● To connect DisplayPort

#### **Procedure**

1. Press **⏻** to turn off the monitor.
2. Hold down **MODE** while pressing **⏻** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Signal Selection” from “Optional Settings”, and press **ENTER**.
4. Select “DisplayPort” with **▲** or **▼**, and press **ENTER**.
5. Select “Sub Pixel Drive” with **▲** or **▼**, and press **ENTER**.
6. Select “On” or “Off” with **▲** or **▼**, and press **ENTER**.
7. Press **RETURN** twice.
8. Select “Finish” with **▲** or **▼**.
9. Press **ENTER**.  
The “Optional Settings” menu is closed.

## 3-4. Enabling/Disabling Mode Selection

Only the modes currently being set can be selected.

This setting is suitable when you are using limited display modes and you do not want to change the customized display state.

### **Procedure**

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Mode Preset” from “Tools”, and press ENTER.
3. Use ▲ or ▼ to select the mode to change, and press ENTER.
4. Select “On” or “Off” with ▲ or ▼.
5. Press ENTER to exit.

---

### **Attention**

- You cannot set all the modes disable. Set at least one mode to “On”.
- 

## 3-5. Setting Language

The language for the Setting menu and messages can be selected.

### **Selectable languages**

English/German/French/Spanish/Italian/Swedish/Japanese/Simplified Chinese/Traditional Chinese

### **Procedure**

1. Select “Menu Settings” from the Adjustment menu, and press ENTER.
2. Select “Language” from “Menu Settings”, and press ENTER.
3. Select a language with ▲ or ▼.
4. Press ENTER to exit.

## 3-6. Setting Adjustment Menu Display

You can change the display position of the Adjustment menu.

### **Procedure**

1. Select “Menu Settings” from the Adjustment menu, and press ENTER.
2. Select “Menu Position” from “Menu Settings”, and press ENTER.
3. Select the position with ▲ or ▼.
4. Press ENTER to exit.

## 3-7. Showing and hiding the EIZO logo

You can choose to show or hide the display of the EIZO logo.

### Procedure

1. Press **⏻** to turn off the monitor.
2. Hold down **MODE** while pressing **⏻** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Logo” from “Optional Settings”, and press **ENTER**.
4. Select “On” or “Off” with **▲** or **▼**, and press **ENTER**.
5. Select “Finish” with **▲** or **▼**.
6. Press **ENTER**.  
The “Optional Settings” menu is closed.

## 3-8. Locking Operation Buttons

The settings can be locked to prevent them being changed.

### Procedure

1. Press **⏻** to turn off the monitor.
2. Hold down **MODE** while pressing **⏻** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Key Lock” from “Optional Settings”, and press **ENTER**.
4. Select “Off”, “Menu”, or “All” with **▲** or **▼**, and press **ENTER**.

Item	Buttons that can be locked
Off (Default setting)	None (All buttons are enabled)
Menu	ENTER button
All	All buttons excluding <b>⏻</b>

5. Select “Finish” with **▲** or **▼**.
6. Press **ENTER**.  
The “Optional Settings” menu is closed.

## 3-9. Enabling/Disabling DDC Communication

The DDC communication can be enabled/disabled. Turn on this setting normally. (This is different from the DDC/CI communication.)

### Procedure

1. Press **⏻** to turn off the monitor.
2. Hold down **MODE** while pressing **⏻** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Connection Config.” from “Optional Settings”, and press **ENTER**.
4. Select “DDC” with **▲** or **▼**, and press **ENTER**.
5. Select “On” or “Off” with **▲** or **▼**, and press **ENTER**.
6. Press **RETURN** to exit.
7. Select “Finish” with **▲** or **▼**.
8. Press **ENTER**.  
The “Optional Settings” menu is closed.

## 3-10. Enabling/Disabling DC5V Output

The DC5V output can be enabled/disabled. Turn off this setting normally.

### Procedure

1. Press **⏻** to turn off the monitor.
2. Hold down **MODE** while pressing **⏻** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Connection Config.” from “Optional Settings”, and press **ENTER**.
4. Select “DC5V Output” with **▲** or **▼**, and press **ENTER**.
5. Select “On” or “Off” with **▲** or **▼**, and press **ENTER**.
6. Press **RETURN** to exit.
7. Select “Finish” with **▲** or **▼**.
8. Press **ENTER**.

The “Optional Settings” menu is closed.

## 3-11. Resetting to Default

There are two different Resets: Mode Reset for only resetting the mode adjustments to the default settings and All Reset for resetting all settings to the defaults.

### Attention

- After resetting, you cannot undo the operation.

### Note

- Regarding the default setting, see “7-5. Main Default Settings” (page 32).

### ● Resetting the mode adjustments

Only the mode adjustment value for the currently selected mode can be reset to the default settings.

#### Procedure

1. Select “Mode” from the Adjustment menu, and press **ENTER**.
2. Select “Mode Reset” from “Mode”, and press **ENTER**.
3. Select “Execute” with **▲** or **▼**.
4. Press **ENTER**.

The mode adjustments revert to the default settings.

### ● Resetting all adjustments to the default settings

All settings are reset to the default settings (except the “Input” and “Optional Settings” menus).

#### Procedure

1. Select “Tools” from the Adjustment menu, and press **ENTER**.
2. Select “All Reset” from “Tools”, and press **ENTER**.
3. Select “Execute” with **▲** or **▼**.
4. Press **ENTER**.

All settings excluding the “Input” and “Optional Settings” menus are reset to the default settings.

# Chapter 4 Controlling Monitor Quality

This product has the built-in Integrated Front Sensor, which allows the user to perform periodic calibration and Grayscale Check for the monitor independently.

The pattern images for checking the quality of the monitor are loaded on the monitor so that the patterns can be checked with an independent monitor.

---

**Attention**

- When a panel protector (RP-901) is installed, the Integrated Front Sensor cannot be used. Do not execute “SelfCalibration” or “Grayscale Check” of “RadiCS SelfQC”. Be sure to set “Meas. Frequency” to “Off”.
- When a panel protector (RP-911) is installed, the Integrated Front Sensor can be used. After installation, be sure to execute correlation using RadiCS/RadiCS LE.

---

**Note**

- To perform advanced quality control that conforms to medical standards, use the optional monitor quality control software kit “RadiCS UX1”.
  - To align the measurement result of the integrated front sensor with that of the UX1 sensor to be set as a reference, execute correlation using RadiCS/RadiCS LE. Periodical correlation enables the measurement accuracy of the integrated front sensor equivalent to that of the UX1 sensor. For details, refer to RadiCS/RadiCS LE User’s Manual.
  - The adjustment target for the calibration, judgment value for the grayscale check, and schedule for both of them can be set using the “RadiCS” software.
  - The adjustment target and schedule for the calibration can be set using the supplied “RadiCS LE” software. For details, refer to RadiCS LE User’s Manual (on the CD-ROM).
  - After changing the orientation of the monitor, be sure to perform correlation.
- 

## 4-1. Executing Calibration

---

SelfCalibration can be executed and latest result can be viewed. In addition, when the schedule is set using RadiCS/RadiCS LE, the next calibration schedule appears.

### ● Executing SelfCalibration

---

**Attention**

- Do not execute this function when a panel protector (RP-901) is installed.
  - After SelfCalibration is executed, execute Grayscale Check under the actual temperature of the monitor usage environment.
- 

**Procedure**

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “SelfCalibration” from “RadiCS SelfQC”, and press ENTER.
3. Select “Execute” with ▲ or ▼.
4. Press ENTER to exit.  
SelfCalibration is executed.

## ● Viewing SelfCalibration result

---

**Attention**

- The result can be viewed when “QC History” is “On” (see “4-4. Setting On/Off for Warning and QC History” (page 20)).
- 

**Procedure**

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “SelfCalibration” from “RadiCS SelfQC”, and press ENTER.
3. Select “Result” with ▲ or ▼.
4. Select “DICOM”, “CAL1” or “CAL2” with ▲ or ▼.
5. Press ENTER to exit.

The SelfCalibration result (Max. Error Rate, Last Time, and Error Code) appears.

## 4-2. Checking Grayscale

---

Grayscale Check can be executed and the latest result can be viewed. In addition, when the schedule is set using RadiCS, the next grayscale check schedule appears.

### ● Executing Grayscale Check

---

**Attention**

- Do not execute this function when a panel protector (RP-901) is installed.
  - Execute Grayscale Check under the actual temperature of the monitor usage environment.
- 

**Procedure**

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “Grayscale Check” from “RadiCS SelfQC”, and press ENTER.
3. Select “Execute” with ▲ or ▼.
4. Press ENTER to exit.

Grayscale Check is executed.

### ● Viewing grayscale check results

---

**Attention**

- The results can be viewed when “QC History” is “On”.
- 

**Procedure**

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “Grayscale Check” from “RadiCS SelfQC”, and press ENTER.
3. Select “Result” with ▲ or ▼.
4. Select “DICOM”, “CAL1” or “CAL2” with ▲ or ▼.
5. Press ENTER to exit.

Up to the five latest grayscale check results (Passed/Failed/Canceled/ERR) appear.

## 4-3. Acquiring Life Prediction Data

The data necessary to predict the monitor life can be acquired.

### Attention

- When a panel protector (RP-901) is installed, be sure to set “Meas. Frequency” to “Off”.

### Note

- Data are collected every 100 hours.
- When you select “Power Save”, data are collected when the mode switches to Power Save or the secondary power off after the above timing is reached.
- When you select “Routine”, data are saved after the following times have elapsed.
  - 500, 1000, 2000, 4000, 7000, 10000, 15000, 20000, 25000, 30000, 35000, 40000, 45000, 50000 hours

### Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “LEA” from “RadiCS SelfQC”, and press ENTER.
3. Select “Meas. Frequency” with ▲ or ▼.
4. Select “Power Save”, “Routine”, or “Off” with ▲ or ▼.
5. Press ENTER to exit.

## 4-4. Setting On/Off for Warning and QC History

The warning and QC history displays for the failure of the Grayscale Check result can be set to on/off.

### Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “Settings” from “RadiCS SelfQC”, and press ENTER.
3. Select “Warning” or “QC History” with ▲ or ▼.
4. Select “On” or “Off” with ▲ or ▼.
5. Press ENTER to exit.

## 4-5. Using Quick Image Checker function

A test pattern is displayed to enable a simplified check of the monitor display. (Quick Image Checker function)

### Attention

- The Quick Image Checker function is not available under the following conditions.
  - RadiCS/RadiCS LE is running (exit RadiCS/RadiCS LE if it is running).
  - Adjustment menu is displayed.
  - Other than the DICOM/CAL1/CAL2 mode is used.
  - “Sub Pixel Drive” is set to “On”.
- The Quick Image Checker function requires several minutes to prepare the patterns immediately after the monitor power is turned on. Please wait for a while.

### Note

- The AAPM test pattern is saved on the monitor by default.
- Only one pattern can be saved on the monitor.
- To change the pattern image, use RadiCS/RadiCS LE. For details, refer to RadiCS/RadiCS LE User’s Manual.

### Procedure

1. Hold down ▲ for two seconds or more.
2. The pattern image is displayed.

# Chapter 5 Power Saving Functions

## 5-1. Setting Power Save

The monitor can be set to the power saving mode according to the PC status. When the monitor has shifted to the power saving mode, the images on the screen are not displayed.

### Attention

- Turning off the main power switch or unplugging the power cord completely shuts off power supply to the monitor.
- Devices connected to the USB downstream port work even when the monitor is in power saving mode or when the power button of the monitor is Off.
- Five seconds before the monitor enters the power saving mode, it displays a preliminary message.

### Procedure

1. Select “PowerManager” from the Adjustment menu, and press ENTER.
2. Select “Power Save” from “PowerManager”, and press ENTER.
3. Select “Off” and either “High” or “Low” with ▲ or ▼.

Setting	Function
High	Turns off the backlight. The power saving effect is the greatest.
Low	Turns down the backlight to the lowest setting. The time until the operation stabilizes after exiting the Power Save mode next time can be reduced.

4. Press ENTER to exit.

### Power Saving System

#### At DVI signal input

This monitor complies with the “DVI DMPM” standard.

#### At DisplayPort signal input

This monitor complies with the “DisplayPort Standard V1.1a”.

The monitor enters the power saving mode after five seconds in connection with the PC setting.

PC	Monitor	Power indicator
On	Operation mode	Green
Power saving mode	Power saving mode	Orange

## 5-2. Adjusting the Brightness of Indicator

The brightness of the power indicator (green) when the screen is displayed can be set. (The power indicator is set to light up when the power is switched on, and set to 4 by default.)

### Procedure

1. Select “PowerManager” from the Adjustment menu, and press ENTER.
2. Select “Indicator” from “PowerManager”, and press ENTER.
3. Select the indicator brightness “Off” or in the range from 1 to 7 with ▲ or ▼ as desired.
4. Press ENTER to exit.

## 5-3. Setting DisplayPort Power Save

---

The power setting of the DisplayPort of the monitor can be switched in the power saving mode.

### **Attention**

- When “On” is set, the display position on the window may not be positioned correctly when the monitor power is turned on or off. In such a case, turn the setting to “Off”.
- 

### **Procedure**

1. Press **⏻** to turn off the monitor.
2. Hold down **MODE** while pressing **⏻** for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Connection Config.” from “Optional Settings”, and press **ENTER**.
4. Select “DP Power Save”, and press **ENTER**.
5. Select “On” or “Off” with **▲** or **▼**, and press **ENTER**.
6. Press **RETURN** to exit.
7. Select “Finish” with **▲** or **▼**.
8. Press **ENTER**.  
The “Optional Settings” menu is closed.

## 5-4. Setting the Power Saving Mode when a Person Leaves the Monitor

The sensor on the front side of the monitor detects the movement of a person by setting Presence Sensor to “Auto” or “Manual”. When a person moves away from the monitor, it shifts to the power saving mode automatically and does not display the images on the screen to reduce the power consumption. When a person comes near the monitor again, it recovers from the power saving mode and displays the images. When “Manual” is selected, the detection area, sensitivity, and time until the power saving mode is activated can be set according to the monitor usage environment and movement of the user.

### Note

- At the time of shifting to power saving mode, the message that indicates shifting is displayed.

### Attention

- This setting cannot be used during “RadiCS SelfQC” execution.
- It also cannot be used during the landscape position display.

### Procedure

1. Select “PowerManager” from the Adjustment menu, and press ENTER.
2. Select “Presence Sensor” from “PowerManager”, and press ENTER.
3. Select “Auto”, “Manual”, or “Off” with ▲ or ▼.
4. Press ENTER to exit.
5. After selecting “Manual”, set the details.

Item	Setting Range	Description
Range	70 cm 90 cm 120 cm	If a person leaves the specified detection area (perpendicular to the screen), the monitor shifts to the power saving mode. The setting ranges (on the left) show rough values.
Time	5, 30 sec 1, 3, 5, 10, 15, 30, 45, 60 min	Specifies the time from when a person leaves the monitor until the “No Presence detected” message appears. The monitor shifts to the power saving mode about 20 seconds after the message is displayed.
Sensitivity	Level 5	Regardless of whether the object is moving or not, the monitor cannot shift to the power saving mode if a person or a still object is present within the detection area. This setting is recommended if the monitor frequently shifts to the power saving, even though a user is in front of the screen.
	Level 4 Level 3 Level 2 Level 1	Since whether the object is moving or not is identified, the monitor can shift to the power saving mode even if there is a chair or wall in the detectable range. When the setting level is high (“Level 4” at maximum), even minor movements of a person are detected, causing the monitor to shift to the power saving mode only infrequently. On the other hand, when the setting level is low (“Level 1” at minimum), only major movements of a person are detected, allowing the monitor to shift to the power saving mode frequently. If the monitor does not shift to the power saving mode or recovers from the mode contrary to your expectations due to phenomenon such as the effect of lightening or ambient light, try decreasing the setting level.
Reset	-	All adjustments are reset to the default settings.

# Chapter 6 Troubleshooting

If a problem still remains after applying the suggested remedies, contact your local EIZO representative.

## 6-1. No Picture

Problem	Possible cause and remedy
<p><b>1. No picture</b></p> <ul style="list-style-type: none"> <li>Power indicator does not light.</li> </ul> <hr/> <ul style="list-style-type: none"> <li>Power indicator is lighting orange and green.</li> </ul> <hr/> <ul style="list-style-type: none"> <li>Power indicator is flashing orange and green.</li> </ul>	<ul style="list-style-type: none"> <li>Check whether the power cord is connected properly.</li> <li>Turn the main power switch on.</li> <li>Press .</li> <li>Turn off the main power, and then turn it on again a few minutes later.</li> </ul> <hr/> <ul style="list-style-type: none"> <li>Switch the input signal (see <a href="#">“Switching among input signals”</a> (page 28)).</li> <li>Move the mouse or press any key on the keyboard.</li> <li>Check whether the PC is turned on.</li> <li>When Presence Sensor is set to “Auto” or “Manual”, the monitor may have shifted to the power saving mode. Come near to the monitor.</li> <li>Turn off the main power, and then turn it on again.</li> </ul> <hr/> <ul style="list-style-type: none"> <li>The device that is connected using DisplayPort have a problem. Solve the problem, turn off the monitor, and then turn it on again. Refer to the User’s Manual of the output device for further details.</li> </ul>
<p><b>2. The message below appears.</b></p> <hr/> <ul style="list-style-type: none"> <li>This message appears when no signal is input. Example:</li> </ul> <div data-bbox="269 1180 673 1453" data-label="Image"> </div> <hr/> <ul style="list-style-type: none"> <li>The message shows that the input signal is out of the specified frequency range. Example:</li> </ul> <div data-bbox="269 1572 673 1919" data-label="Image"> </div> <p>fD: Dot clock fH: Horizontal scan frequency fV: Vertical scan frequency</p>	<p>This message appears when the signal is not input correctly even when the monitor functions properly.</p> <hr/> <ul style="list-style-type: none"> <li>The message shown left may appear, because some PCs do not output the signal soon after power-on.</li> <li>Check whether the PC is turned on.</li> <li>Check whether the signal cable is connected properly.</li> <li>Switch the input signal (see <a href="#">“Switching among input signals”</a> (page 28)).</li> </ul> <hr/> <ul style="list-style-type: none"> <li>Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see “Compatible Resolutions” in the “Instructions for Use”).</li> <li>Reboot the PC.</li> <li>Select an appropriate setting using the graphics board’s utility. Refer to the manual of the graphics board for details.</li> </ul>

## 6-2. Imaging Problems

Problem	Possible cause and remedy
1. The screen is too bright or too dark.	<ul style="list-style-type: none"><li>• Use “Brightness” in the Adjustment menu to adjust it. (The LCD monitor backlight has a limited life span. When the screen becomes dark or begins to flicker, contact your local EIZO representative.)</li></ul>
2. Characters are blurred.	<ul style="list-style-type: none"><li>• Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see “Compatible Resolutions” in the “Instructions for Use”).</li></ul>
3. Afterimages appear.	<ul style="list-style-type: none"><li>• Afterimages are particular to LCD monitors. Avoid displaying the same image for a long time.</li><li>• Use the screen saver or power saving function to avoid displaying the same image for extended periods of time.</li></ul>
4. Interference patterns or pressure marks remain on the screen.	<ul style="list-style-type: none"><li>• Leave the monitor with a white or black screen. The symptom may disappear.</li></ul>

## 6-3. Other Problems

Problem	Possible cause and remedy
1. The adjustment menu does not appear.	<ul style="list-style-type: none"> <li>• Check whether the operation lock function is active (see <a href="#">“3-8. Locking Operation Buttons” (page 16)</a>).</li> </ul>
2. No mode menu is displayed.	<ul style="list-style-type: none"> <li>• Check whether the operation lock function is active (see <a href="#">“3-8. Locking Operation Buttons” (page 16)</a>).</li> </ul>
3. (When using the presence sensor) The images remain to be displayed when you move away from the monitor. / The images do not reappear when you come near it.	<ul style="list-style-type: none"> <li>• Check the setting environment of the monitor. Presence Sensor may not work correctly in the following environments.               <ul style="list-style-type: none"> <li>- There is an object that reflects light such as a mirror or glass in front of the sensor.</li> <li>- The monitor is located in a place that receives direct sunlight.</li> <li>- There is an infrared light emitting device near the monitor.</li> <li>- There is some obstacle in front of the sensor.</li> </ul> </li> <li>• Check whether any dirt on the sensor. Clean the sensor with a soft cloth.</li> <li>• Check whether you are in front of the monitor. The infrared ray is emitted perpendicularly from the sensor to detect a person.</li> </ul>
4. The monitor connected with the USB cable is not detected. / USB devices connected to the monitor does not work.	<ul style="list-style-type: none"> <li>• Check whether the USB cable is connected correctly (see <a href="#">“7-3. Making Use of USB (Universal Serial Bus)” (page 30)</a>).</li> <li>• Change the USB port to another one. If the PC or peripheral devices works correctly by changing the USB port, contact your local EIZO representative. (Refer to the manual of the PC for details.)</li> <li>• Reboot the PC.</li> <li>• If the peripheral devices work correctly when the PC and peripheral devices are connected directly, please contact your local EIZO representative.</li> <li>• Check whether the PC and its OS are USB compliant. (For USB compliance of the respective devices, consult their manufacturers.)</li> <li>• For Windows, check the PC’s BIOS setting for USB. (Refer to the manual of the PC for details.)</li> </ul>
5. Integrated Front Sensor stays out.	<ul style="list-style-type: none"> <li>• Turn the main power off and on again.</li> </ul>
6. SelfCalibration/Grayscale Check failure	<ul style="list-style-type: none"> <li>• Refer to the error code table in the “Instructions for Use”.</li> <li>• If an error code that does not appear in the error code table is displayed, contact your local EIZO representative.</li> </ul>
7. Time is not displayed correctly in the history of SelfCalibration/Grayscale Check	<ul style="list-style-type: none"> <li>• Execute Monitor Detection using RadiCS/RadiCS LE. For details, refer to RadiCS/RadiCS LE User’s Manual.</li> </ul>

# Chapter 7 Reference

## 7-1. Attaching Arm

An optional arm (or an optional stand) can be attached by removing the stand section. Please refer to our web site for the corresponding optional arm (or optional stand). <http://www.eizoglobal.com>

### Attention

- When attaching an arm or stand, follow the instructions of their User's Manual.
- When using another manufacturer's arm or stand, confirm the following in advance and select one conforming to the VESA standard.
  - Clearance between the screw holes: 100 mm × 100 mm
  - Strong enough to support weight of the monitor unit (except the stand) and attachments such as cables.
- When using another manufacturer's arm or stand, use the following screws to secure it.
  - Screws fixing the stand to the monitor and screws supplied with this monitor
- When using an arm or stand, attach it to meet the following tilt angles of the monitor.
  - Up 45°, down 45°
- Connect the cables after attaching an arm or stand.
- Do not move the removed stand up and down. Doing so may result in injury or device damage.
- The monitor and arm or stand are heavy. Dropping them may result in injury or equipment damage.

### Procedure


**1. Lay the LCD monitor on a soft cloth spread over on a stable surface with the panel surface facing down.**

**2. Remove the stand.**

Prepare a screwdriver. Loosen the screws (four) securing the unit and the stand with the screwdriver.

**3. Attach the arm or stand to the monitor.**


Secure the monitor to the arm or stand using the screws specified in the User's Manual of the arm or stand.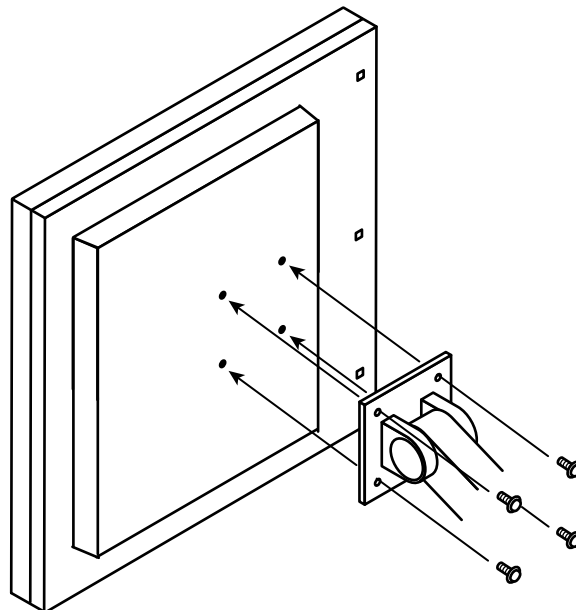

## 7-2. Connecting Multiple PCs

The product has multiple connections to PCs and allows you to switch to one of the connections for display.

### Connection examples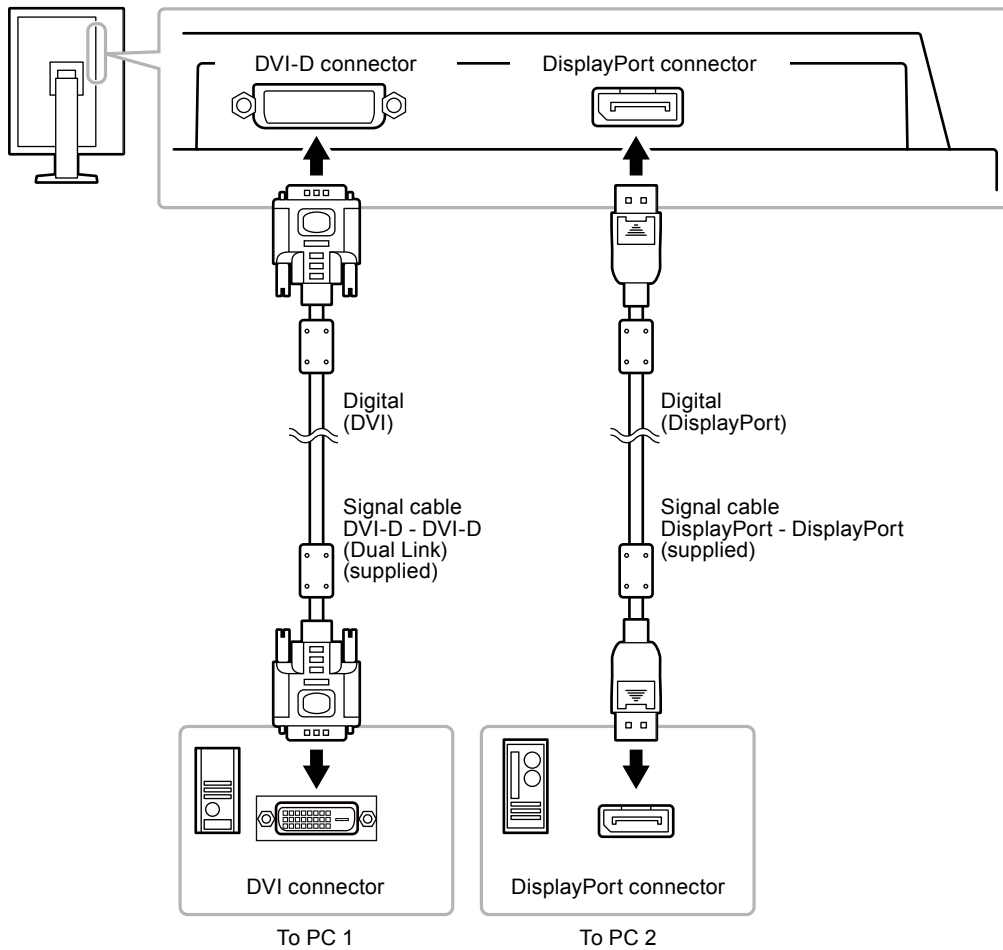


### ● Switching among input signals

#### Procedure

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Input” from “Tools”, and press ENTER.
3. Select “DVI” or “DisplayPort” with ▲ or ▼.
4. Press ENTER to exit.

When the signal is switched, the active input port name appears at the top right corner of the screen.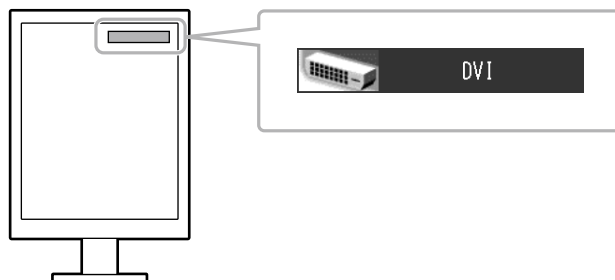

## ● Setting input signal switching procedure

Setting	Function
Auto	The monitor recognizes the connector through which PC signals are input. When a PC is turned off or enters the power saving mode, the monitor automatically displays another signal.
Manual	The monitor displays the signal from the selected connector despite the signal input. Select an active input signal with "Input".

### **Procedure**

1. Select "Tools" from the Adjustment menu, and press ENTER.
2. Select "Input Selection" from "Tools", and press ENTER.
3. Select "Auto" or "Manual" with ▲ or ▼.
4. Press ENTER to exit.

---

### **Note**

- If you select "Auto", the monitor enters the power saving mode after all the PCs have entered the power saving mode or have been switched off.
-

## 7-3. Making Use of USB (Universal Serial Bus)

This monitor has a USB hub. It works as a USB hub when connected to a USB-compatible PC, allowing the connection of peripheral USB devices.

### ● Required system environment

- A PC equipped with a USB port
- Windows 2000/XP/Vista/7/8 or Mac OS 9.2.2 and Mac OS X 10.2 or later
- USB cable

#### **Attention**


- This monitor may not work depending on PC, OS or peripheral devices to be used. For USB compatibility of peripheral devices, contact their manufacturers.
- Devices connected to the USB downstream port work when the monitor is in power saving mode or when the power button of the monitor is Off.
- When the main power of the monitor is Off, a device connected to the USB downstream port will not operate.

### ● Connection procedure (Setup of USB function)

1. Connect the monitor first to a PC using the signal cable, and run the PC.
2. Connect the USB cable between the USB downstream port of the PC and the USB upstream port of the monitor.

The USB hub function is set up automatically upon connection of the USB cable.

3. Connect the peripheral USB devices to the USB downstream port of the monitor.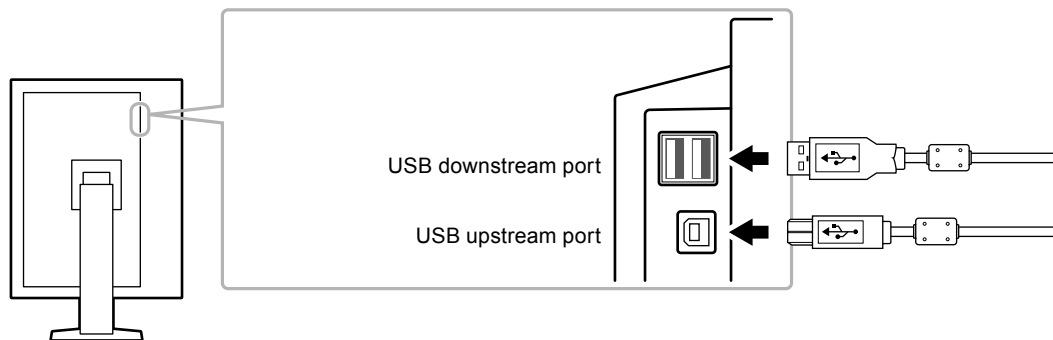

## 7-4. Showing Monitor Information

### ● Displaying signal information


The information about the current input signals can be displayed.

#### **Procedure**

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Signal Info” from “Tools”, and press ENTER.

The “Signal Info” appears.

(Example)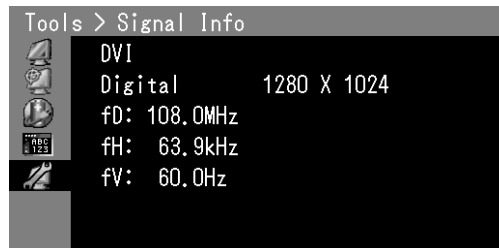

### ● Showing monitor information

The information about the monitor can be displayed.


#### **Procedure**

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Monitor Info” from “Tools”, and press ENTER.

The “Monitor Info” appears.

The model name, serial number, firmware version, and usage time can be displayed.

(Example)

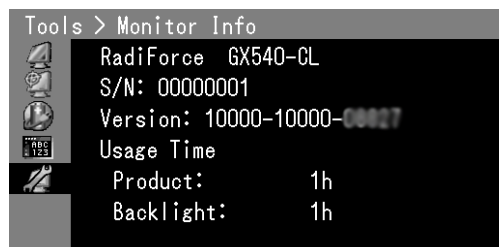---

#### **Attention**

- When you turn on the monitor for the first time, the usage time may be more than 0 due to a factory inspection.
-

## 7-5. Main Default Settings

**CAL Switch Mode:** The default display setting is 1-DICOM.

Mode	Brightness	Gamma value
1-DICOM	500 cd/m <sup>2</sup>	DICOM
2-CAL1	400 cd/m <sup>2</sup>	DICOM
3-CAL2	300 cd/m <sup>2</sup>	DICOM

### Others

Input* <sup>1</sup>	DVI
Presence Sensor	Off
Power Save	High
DP Power Save* <sup>1</sup>	Off
Input Selection	Auto
Logo* <sup>1</sup>	Off
Key Lock* <sup>1</sup>	Off
DDC* <sup>1</sup>	On
DC5V Output* <sup>1</sup>	Off
Orientation* <sup>1</sup>	Portrait(HW)
Grayscale Warning	On
Signal Format* <sup>1</sup>	Standard
Sub Pixel Drive* <sup>1</sup>	Off
Menu Position	Center
Language	English

\*1 These menus cannot be initialized with "All Reset" (see [page 17](#)).

# Appendix

## LIMITED WARRANTY

### LIMITED WARRANTY

EIZO Corporation (hereinafter referred to as “**EIZO**”) and distributors authorized by EIZO (hereinafter referred to as the “**Distributors**”) warrant, subject to and in accordance with the terms of this limited warranty (hereinafter referred to as the “**Warranty**”), to the original purchaser (hereinafter referred to as the “**Original Purchaser**”) who purchased the product specified in this document (hereinafter referred to as the “**Product**”) from EIZO or Distributors, that EIZO and Distributors shall, at their sole discretion, either repair or replace the Product at no charge if the Original Purchaser becomes aware within the Warranty Period (defined below) that (i) the Product malfunctions or is damaged in the course of normal use of the Product in accordance with the description in the instruction manual attached to the Product (hereinafter referred to as the “**User’s Manual**”), or (ii) the LCD panel and brightness of the Product cannot maintain the recommended brightness specified in the User’s Manual in the course of normal use of the Product in accordance with the description of the User’s Manual.

The period of this Warranty is five (5) years from the date of purchase of the Product (hereinafter referred to as the “Warranty Period”), provided that the brightness of the Product shall be warranted only if the Product has been used within the recommended brightness described in the User’s Manual and the Warranty Period for the brightness is limited to five (5) years from the date of purchase of the Product. EIZO and Distributors shall bear no liability or obligation with regard to the Product in relation to the Original Purchaser or any third parties other than as provided under this Warranty.

EIZO and Distributors will cease to hold or store any parts of the Product upon expiration of seven (7) years after the production of such parts is discontinued. In repairing the monitor, EIZO and Distributors will use renewal parts which comply with our QC standards.

The Warranty is valid only in the countries or territories where the Distributors are located. The Warranty does not restrict any legal rights of the Original Purchaser.

Notwithstanding any other provision of this Warranty, EIZO and Distributors shall have no obligation under this Warranty whatsoever in any of the cases as set forth below:

- (a) Any defect of the Product caused by freight damage, modification, alteration, abuse, misuse, accident, incorrect installation, disaster, faulty maintenance and/or improper repair by third party other than EIZO and Distributors;
- (b) Any incompatibility of the Product due to possible technical innovations and/or regulations;
- (c) Any deterioration of the sensor;
- (d) Any deterioration of display performance caused by the deterioration of expendable parts such as the LCD panel and/or backlight, etc. (e.g. changes in color, changes in color uniformity, defects in pixels including burnt pixels, etc.);
- (e) Any deterioration of the Product caused by the use at higher brightness than the recommended brightness described in the User’s Manual;
- (f) Any defect of the Product caused by external equipment;
- (g) Any defect of the Product on which the original serial number has been altered or removed;
- (h) Any normal deterioration of the product, particularly that of consumables, accessories, and/or attachments (e.g. buttons, rotating parts, cables, User’s Manual, etc.); and
- (i) Any deformation, discoloration, and/or warp of the exterior of the product including that of the surface of the LCD panel.

To obtain service under the Warranty, the Original Purchaser must deliver the Product, freight prepaid, in its original package or other adequate package affording an equal degree of protection, assuming the risk of damage and/or loss in transit, to the local Distributor. The Original Purchaser must present proof of purchase of the Product and the date of such purchase when requesting services under the Warranty.

The Warranty Period for any replaced and/or repaired product under this Warranty shall expire at the end of the original Warranty Period.

EIZO OR DISTRIBUTORS ARE NOT RESPONSIBLE FOR ANY DAMAGE TO, OR LOSS OF, DATA OR OTHER INFORMATION STORED IN ANY MEDIA OR ANY PART OF ANY PRODUCT RETURNED TO EIZO OR DISTRIBUTORS FOR REPAIR.

EIZO AND DISTRIBUTORS MAKE NO FURTHER WARRANTIES, EXPRESSED OR IMPLIED, WITH RESPECT TO THE PRODUCT AND ITS QUALITY, PERFORMANCE, MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR USE. IN NO EVENT SHALL EIZO OR DISTRIBUTORS BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, CONSEQUENTIAL OR OTHER DAMAGE WHATSOEVER (INCLUDING, WITHOUT LIMITATION, DAMAGES FOR LOSS OF PROFIT, BUSINESS INTERRUPTION, LOSS OF BUSINESS INFORMATION, OR ANY OTHER PECUNIARY LOSS) ARISING OUT OF THE USE OR INABILITY TO USE THE PRODUCT OR IN ANY CONNECTION WITH THE PRODUCT, WHETHER BASED ON CONTRACT, TORT, NEGLIGENCE, STRICT LIABILITY OR OTHERWISE, EVEN IF EIZO OR DISTRIBUTORS HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. THIS EXCLUSION ALSO INCLUDES ANY LIABILITY WHICH MAY ARISE OUT OF THIRD PARTY CLAIMS AGAINST THE ORIGINAL PURCHASER. THE ESSENCE OF THIS PROVISION IS TO LIMIT THE POTENTIAL LIABILITY OF EIZO AND DISTRIBUTORS ARISING OUT OF THIS LIMITED WARRANTY AND/OR SALES.

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---

## Recycling Information

This product, when disposed of, is supposed to be collected and recycled according to your country's legislation to reduce environmental burden. When you dispose of this product, please contact a distributor or an affiliate in your country.

The contact addressees are listed on the EIZO website below.

<http://www.eizoglobal.com>

For recycling information for customers in Switzerland, please refer to the following website.

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